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Instructions for use (EN,FR,IT,DA,DE)

23330906 Version 3

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved.©

2023-09-28

EN

The product is intended to be used as a skin protectant in the management of skin damage in skin folds and/or other skin-to-skin contact areas.

Indication

The product is indicated for use on skin between touching skin surfaces where inflammation and/or irritation is seen or anticipated due to perspiration and friction. Symptoms may be termed intertrigo.

Intertrigo (intertriginous dermatitis) is a form of moisture associated skin damage and is commonly found in body regions such as armpits, under the breasts, abdomen, groins, and between fingers or toes.

Contraindication

The use of the product by children and pregnant or nursing women has not been investigated. Use of the product by this population is not justified due to lack of data. Consult a health care professional for further advice.

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Information

The product is a non-sterile polyester fabric that reduces friction between skin folds and wicks away moisture. A silver complex in the fabric provides an antimicrobial effect against microorganisms commonly found in skin folds. The antimicrobial activity may also help reduce odour.

If the skin condition does not improve within 4 weeks of use, a health care professional should assess if treatment should be continued.

If you experience a suspected allergic reaction (such as rash, itching, redness or blistering) or any other side-effects, please contact your health care professional.

The product can be left in place during MRI (Magnetic Resonance Imaging) examination. It is compatible with up to 3 Tesla.

Use of any powders, ointments, creams or emollients with the product may prevent it from working as intended.

The product should be stored at room temperature in dry conditions in its original packaging until use.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Caution

The product should not be applied directly on wounds.

The product should not be used on patients with a known sensitivity to silver.

In the event of an infection, use of the product does not replace the need for adequate infection treatment according to institutional guidelines.

The product should be removed prior to radiation treatment or examinations that include X-rays, ultrasonic treatment, diathermy or microwaves.

The product should not be washed.

The product should not be worn in the shower or when bathing.

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How to use Preparation

Gently cleanse the skin and pat the area dry without rubbing.

Application

Measure and cut off an appropriately sized piece of fabric with a clean pair of scissors. Allow for a minimum of 5 cm of fabric to be exposed outside the skin fold to help moisture evaporate.

Place a single layer of fabric in the skin fold, ensuring that the edge of the fabric is aligned with the base of the skin fold.

Gently smooth the fabric over the skin to ensure an even surface. Leave at least 5 cm of fabric exposed outside the skin fold. Secure the fabric with the skin fold or tucked under clothing.

Each piece of fabric can be used for a maximum of 5 days depending on soiling, odour, amount of moisture and general skin condition. However, if the fabric is removed within 5 days, e.g. during showering or bathing, it should not be reapplied and a fresh piece of fabric should be used.

Removal

Remove the fabric from skin folds by separating the skin fold and gently lifting the fabric away.

Disposal

The product should be disposed of in accordance with local guidelines, e.g. with normal household waste.

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